

# Measurement Chart for VetGood Slim Boots





## Getting the right size for your pet.

1. Place sizing chart on firm surface.
2. Place paw on sizing chart and align the edge of paw with **bold line labelled A**
3. Lift pet's opposite paw so that weight is placed firmly onto paw being measured.
4. Mark BOTH SIDES of weighted paw at its widest point on the sheet of paper.
5. Measure the distance and choose correct boot size (if you are between sizes, choose the smaller size).

AVAILABLE SIZES						
2in X-Small (Yellow)	Coming Soon X-Small 2 (Pink)	Coming Soon Short (Orange)	3in Small (Blue)	3.25 Medium (Red)	3.75in Large (Green)	4.5in X-Large (Purple)

First, print the Sizing Chart at 100% size. After printing, verify the accuracy of this chart with a ruler before measuring.

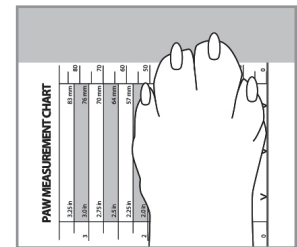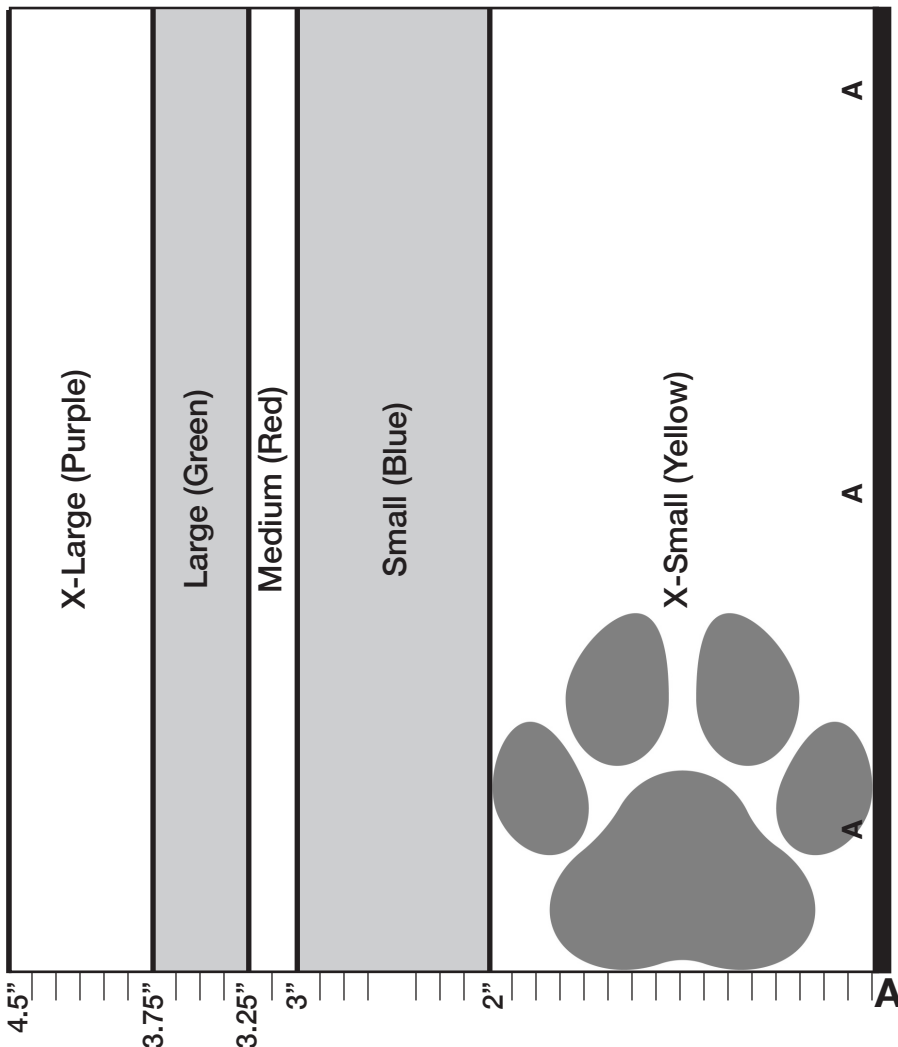

### Questions?

Call or email us at  
[info@vetgood.com](mailto:info@vetgood.com)  
 P: 800.245.3413  
 F: 646.783.3808

[www.vetgood.com](http://www.vetgood.com)

Protective Boot  
 U.S. Patent No.  
 D747,048 S